

Dressing change



Stop the PICO pump by pressing the orange button. Remove the pump from the dressing by untwisting the connectors. **NOTE: if using extension tubing do not discard as this can be used for the dressing change.**

Remove the PICO dressing by stretching the fixation strips away from the skin. Lift the dressing at one corner and peel back until it has been fully removed.



The PICO dressing and fixation strips should be disposed of as clinical waste.

At the end of therapy the batteries should be removed from the PICO 7 pump; both batteries and the pump should be disposed of according to local regulations.

Advanced Wound Management Customer Care Center

Smith+Nephew
Fort Worth, TX 76109
USA

T 800 876-1261
F 727 392-6914

©2022 Smith+Nephew
® Trademark of Smith+Nephew
737-5171-20221017

www.possiblewithpico.com

www.smith-nephew.com



Watch Application Video

Smith+Nephew does not provide medical advice. It is the responsibility of clinicians to determine and utilize products and techniques appropriate for each of their individual patients. This is not the full application instructions, for more details please refer to the Instructions for Use (IFU). For detailed product information, including indications for use, contraindications, precautions and warnings, including the magnet warning, please consult the product's applicable IFU prior to use.