


## Breast application using the PICO<sup>o</sup> 7Y System

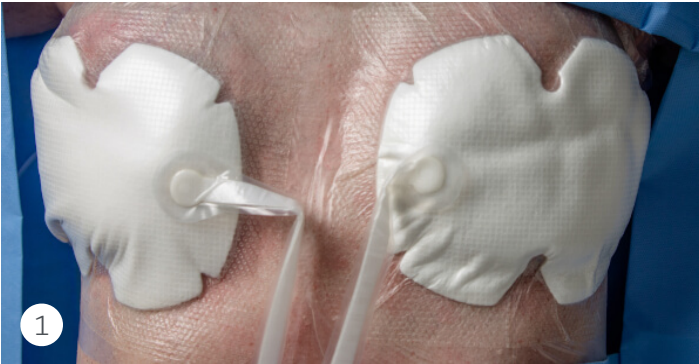

The PICO 7Y System may be used to treat two incisions simultaneously. Clean and prepare the incision according to your local protocol. The dressings can be placed either vertically or horizontally.



Remove the backing from the first PICO dressing and apply over the incision. Port position: uppermost from incision. NOTE: ACTICOAT<sup>®</sup> FLEX may be applied in conjunction with PICO SNPWT.\*



PICO may be used in conjunction with surgical drains, provided the dressing is not placed over tubing where it exits the skin. Place dressing approximately 1 inch above the drains, with the port position uppermost from the incision. Silicone boarder can be cut to fit around drains.



After applying the first dressing, connect the pump to the dressing by twisting the two halves of the connector together, insert the batteries and press the orange button to start therapy. Place a finger over the other port. Once you've established a seal, apply the fixation strips. Once complete, repeat these steps for the second dressing.

### Advanced Wound Management Customer Care Center

Smith+Nephew  
Fort Worth, TX 76109  
USA

T 800 876-1261  
F 727 392-6914

©2022 Smith+Nephew  
\*Trademark of Smith+Nephew  
737-5172-20221017

[www.possiblewithpico.com](http://www.possiblewithpico.com)

[www.smith-nephew.com](http://www.smith-nephew.com)



[Watch Application Video](#)

Smith+Nephew does not provide medical advice. It is the responsibility of clinicians to determine and utilize products and techniques appropriate for each of their individual patients. This is not the full application instructions, for more details please refer to the Instructions for Use (IFU). For detailed product information, including indications for use, contraindications, precautions and warnings, including the magnet warning, please consult the product's applicable IFU prior to use.

\*ACTICOAT FLEX 3 and ACTICOAT FLEX 7 are only approved for use with NPWT for up to 3 days.

Please note that PICO 7Y can be used in conjunction with Pressure Garment Therapy.