

## C-section application



Prepare the site. Periwound skin should be dried completely to promote a good seal.



Peel off the first release handle and place dressing centrally over the wound. The soft port should be uppermost from the wound (if possible) and not over the wound.



Connect the pump to the dressing by twisting the two halves of the connector together, insert the batteries and press the orange button to start therapy. The pump will evacuate the air from the dressing and the dressing will become firm.



Apply the fixation strips all around the dressing border as they help maintain the seal. Place each strip so that it overlaps the dressing by approximately 1cm.

### Advanced Wound Management Customer Care Center

Smith+Nephew  
Fort Worth, TX 76109  
USA

T 800 876-1261  
F 727 392-6914

©2022 Smith+Nephew  
® Trademark of Smith+Nephew  
737-5168-20221017

[www.possiblewithpico.com](http://www.possiblewithpico.com)

[www.smith-nephew.com](http://www.smith-nephew.com)



Watch Application Video

Smith+Nephew does not provide medical advice. It is the responsibility of clinicians to determine and utilize products and techniques appropriate for each of their individual patients. This is not the full application instructions, for more details please refer to the Instructions for Use (IFU). For detailed product information, including indications for use, contraindications, precautions and warnings, including the magnet warning, please consult the product's applicable IFU prior to use.

\*ACTICOAT FLEX 3 and ACTICOAT FLEX 7 are only approved for use with NPWT for up to 3 days.