


Hip incision application



Prepare the site. Periwound skin should be dried completely to promote a good seal. PICO dressings may be applied over incisions closed with a variety of methods: Sutures, staples, skin glues, or other surgical skin closures.

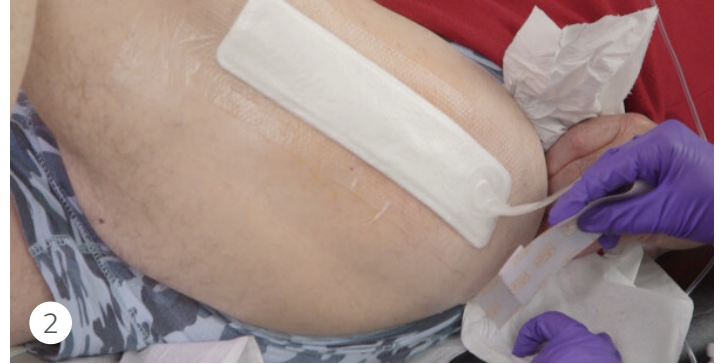

Peel off the first release handle and place dressing centrally over the wound. The soft port should be uppermost from the wound (if possible) and not over the wound.



Remove the remaining handles and smooth dressing around the wound to prevent creasing. Connect the pump to the dressing by twisting the two halves of the connector together, insert the batteries and press the orange button to start therapy.



Assist the dressing to get good contact with the wound/skin by following the dressing with your hand while the pump is evacuating air from the dressing. Once the negative pressure has been successfully delivered, apply the fixation strips to the border of the dressing. The strips may be cut to make them easier to apply.

Advanced Wound Management Customer Care Center

Smith+Nephew
Fort Worth, TX 76109
USA

T 800 876-1261
F 727 392-6914

©2022 Smith+Nephew
® Trademark of Smith+Nephew
737-5169-20221017

www.possiblewithpico.com

www.smith-nephew.com



Watch Application Video

Smith+Nephew does not provide medical advice. It is the responsibility of clinicians to determine and utilize products and techniques appropriate for each of their individual patients. This is not the full application instructions, for more details please refer to the Instructions for Use (IFU). For detailed product information, including indications for use, contraindications, precautions and warnings, including the magnet warning, please consult the product's applicable IFU prior to use.

*ACTICOAT FLEX 3 and ACTICOAT FLEX 7 are only approved for use with NPWT for up to 3 days.