

The PICO System

Smith+Nephew

Troubleshooting Guide - What to do if you have an Air Leak Alarm

PICO[◇] 7

Single Use Negative Pressure
Wound Therapy System

1



An air leak on the pump is the second from left indicator that flashes orange.

2




Double-check the tubing connections are sealed tight. Unscrew and reconnect the tubing at the base of the pump as well as near the white tubing of the dressing.

3



Rub your finger around the silicone edges of the dressing on all four sides. If you see any creases, rub them flat.

4

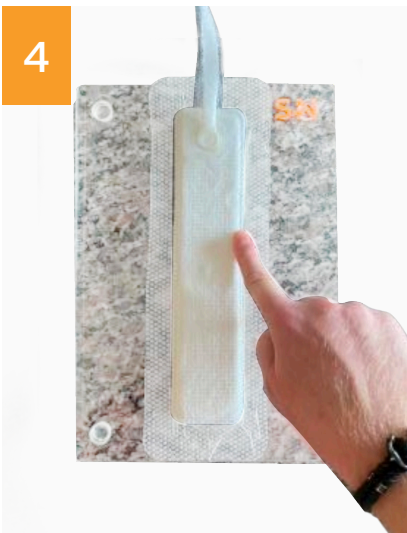

Repeat the process by rubbing the edge of the white foam on all four sides.

5



Press the orange button to restart therapy. You will hear a slight buzzing sound and the pump will vibrate and the green 'OK' and orange 'air leak' light will flash as pressure is re-established. This can take up to 65 seconds. You can also rub your fingers around the four sides of the dressing.

6



If the green 'OK' light is flashing a seal has been achieved. If the 'air leak' light is still flashing please call Customer Care

1-800-876-1261